



CUSTOMIZE YOUR OWN TEMPORARY DENTURES



Materials.



**Add 2 pink medallions to boiling hot water.
(Place in hot water as needed to keep the material soft).**



Take the soften medallions, roll them up and place them on the teeth to create the gums by taking the impression directly into mouth or the stone model.



Fit the material around your gums to shape your dentures by pushing toward the pallet of the mouth or stone model.

NON TOXIC/SAFE TO USE USER FRIENDLY



Once you think it fits securely and comfortably in your mouth, take it out and shape the excess material to create a more natural look.



To polish it and finish the model, deep it back in hot water as need, or if needed to meet your expectations.



Even out the edges with your fingers and thumbs as needed for comfort and cosmetic.



Once your costume made dentures are done, and if you think that they still need to fit better. Place it back in hot water and to mold it again until it meets your expectations.



 **3120 RIVER BRANCH CIRCLE,
KISSIMMEE, FL 34741**

 **@JESSEDENTLLC**

 **JESSEDENTLLC**

 **WWW.JESSEDENTLLC.COM**